

How to Do “System Upgrade” for Skyla VB1

1. Get the new “System Upgrade File” in the USB Drive, from your agent.

Filename as : VB1_yyyymmdd_xxxxxx_ASzz.bin



2. Plug the USB drive to the **USB** locate at the rear panel of VB1 analyzer.



3. Upgrade the System Software of the analyzer


(1) Upgrade the analyzer :

Please back to HOME menu of LCD screen first,

Operate touch screen: **//HOME/SETTINGS/Admin...Settings/System Upgrade**

Please note, after USB drive plugged, it will take few seconds to be recognized.

If VB1 recognized the USB drive successfully, a **USB drive logo** will be hint on screen. then click **YES** to proceed...

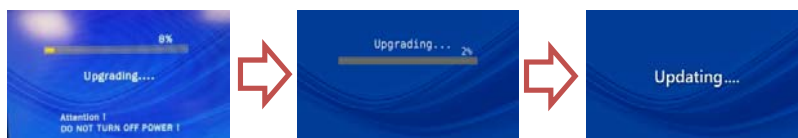

then reboot automatically...



(2) It will take about **7 min** for the upgrade, then system will re-boot after upgraded.

Reboot takes another **3 min or more** depends on any database reorganization internally.

(3) Please make sure **Power line is always ON** during upgradation and re-boot.

(4) Check the **Updated Software Version** of analyzer again :

Operate LCD touch screen **//HOME/SETTINGS/Version**

Software Version : ASzz (new)

Assay Parameter Version : AESww (new)

This is to confirm the software version is up-to-date now.

(5) If there is any error, Error Code maybe with message will be shown on screen.

Report the Error Code to your agent for troubleshooting.

(6) System Upgrade will NOT erase historic analysis results

PS: Universal Password is **5787722** all the time if password forget.

If you do NOT enter any number during the Password changing, the Password function will be temporarily disable until you change Password again with numbers.